

TUTORIEL C2CARE

Installation guide for C2Care Applications on Oculus Quest



Do not expose the headset lenses to the sun.

Installing Meta Quest app and developer mode on your phone/tablet


If you have received your Meta Quest 2 and have already :

- 1 Installed the Meta Quest app on your phone
- 2 Created a Meta account
- 3 Paired your Meta Quest 2 to your app


Go to 4. [Enabling Developer Mode](#), otherwise follow the [steps 1 - 3](#) :

1 Open [Meta Quest \(Oculus\)](#) app on your smartphone or tablet. If you don't have it, download it on the Play Store or Apple Store.


2 After downloading and opening the app, create an account using one of the following options.

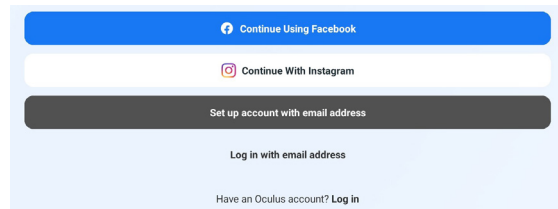



3 Pair your Meta Quest 2 with the code display in your headset or it could be automatically detected.

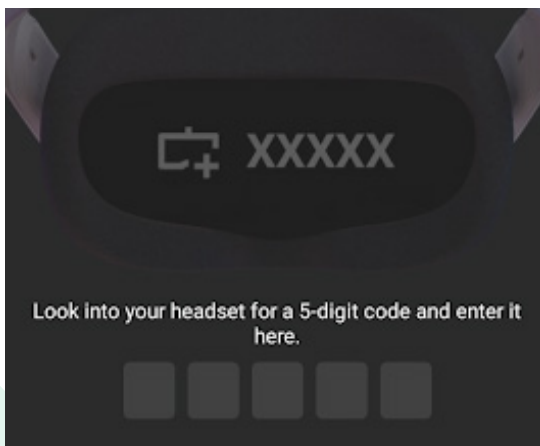


4 Enabling Developer Mode : Go to [Menu](#) and select [Devices](#).

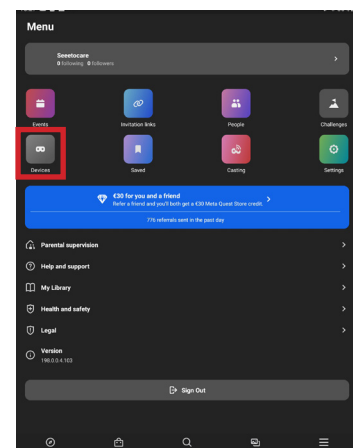


Select **Developer Mode**, then click on **Get started**.

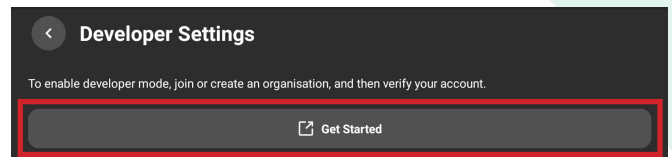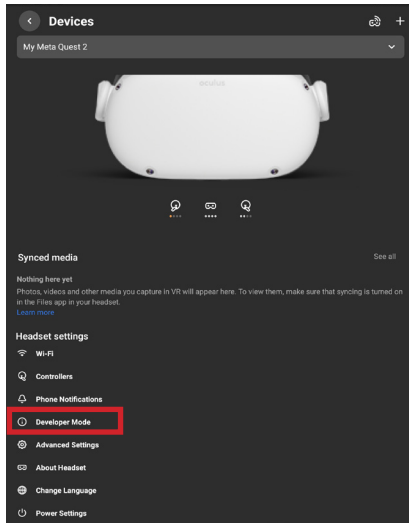


Once on the redirected page, follow the link indicated below.

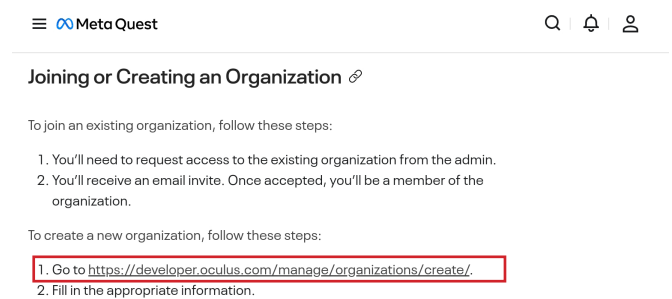

Verify your Developer account by clicking on the **Set Up** button.

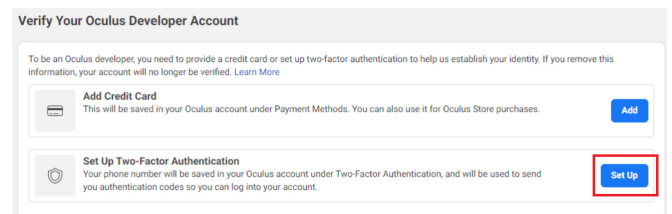


You will be redirected to a **Password and security** page, click on **Edit** at the two-factor authentication level.

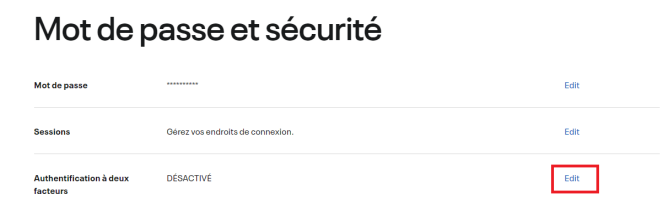

Press **Add Phone Number** and enter the verification code received on your phone.




Return to the tab in step 7 and click **Back to Home**.

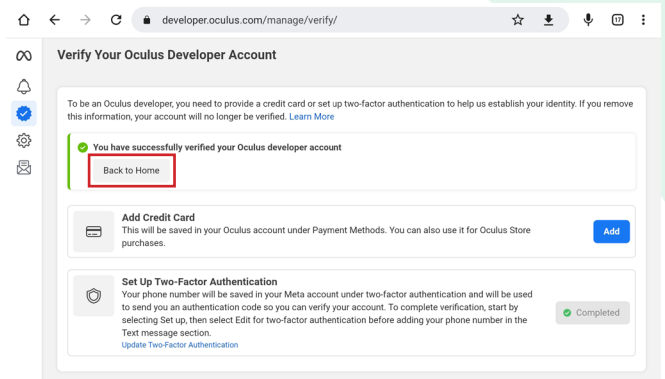


Create your organization:

- 1 Enter the name of your choice
- 2 Accept the conditions
- 3 Press **Submit**

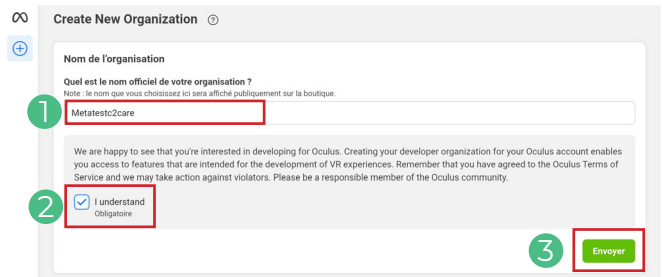

Go back to the **Meta Quest** application and enable the **Developer Mode** as displayed below.

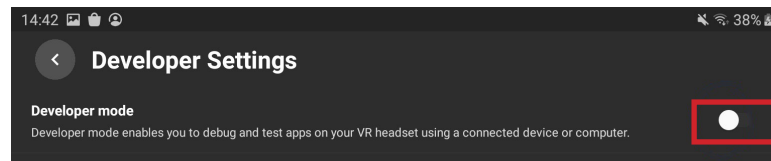

Installing Drivers and SideQuest on your computer

Now you will have to install the drivers (this manipulation is only necessary if you are on **Windows**, otherwise go to **step 5**). To do this, go to the following address:

<https://developer.oculus.com/downloads/package/oculus-go-ADB-drivers/> and click on the **Download** button.

Oculus Go ADB Drivers

VERSION

DOWNLOAD


Published: May 9, 2018

Select the download folder from your computer. Right-click on the folder and select **Extract here**. Now you can open the file you just unzipped by double clicking on it.



3


Once opened, right click on the **android-winsub.inf** file and click **Install**.

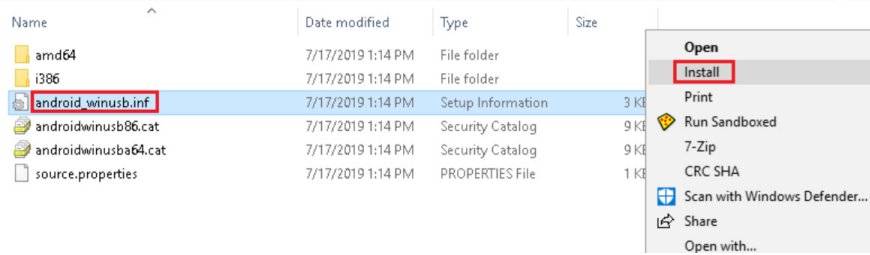

If your computer is under **Windows 11** you will have a few more steps to find the button allowing you to install android_winsub.

- 1 Right click on **android_winsub.inf**
- 2 Click on **Show more options**
- 3 Click on **Install**

4

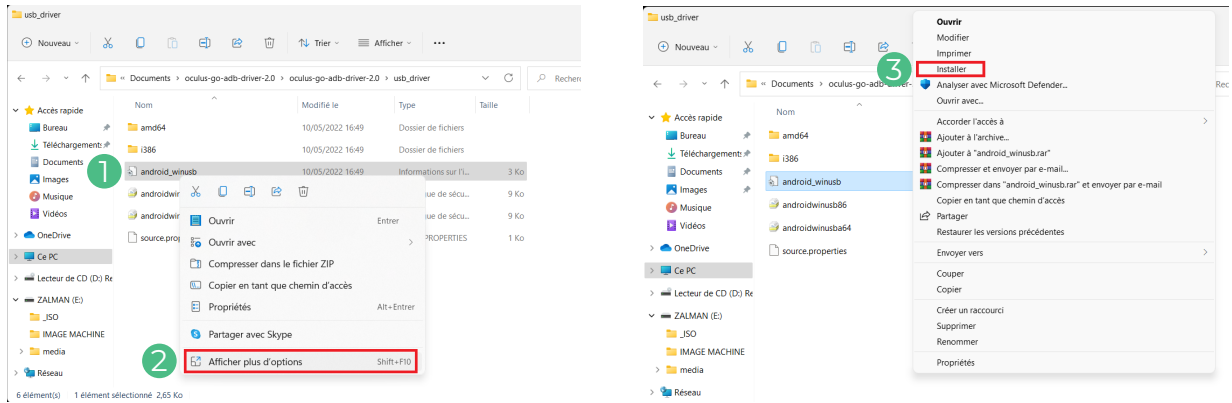

5

Go to <https://sidequestvr.com/setup-howto> to download and install **SideQuest Advanced Installer**.

Advanced Installer (LTS)

Step 1: Download SideQuest & Sign Up
Install SideQuest on windows, linux or mac and create an account [here](#).

Desktop Download

- DOWNLOAD FOR WINDOWS 10 X64** 71.59MB
v0.10.27 / 71.59MB / 680,090 downloads
- DOWNLOAD FOR OS X / MACOS 10.12+** 102.36MB
v0.10.27 / 102.36MB / 113783 downloads
- DOWNLOAD FOR OS X / MACOS 10.12+ (M1/ARM)** 104.22MB
v0.10.27 / 104.22MB / 62373 downloads
- DOWNLOAD FOR LINUX** 67.25MB
v0.10.27 / 67.25MB / 82688 downloads



Downloading and installing C2Care applications

The C2Care applications (in .SDK) are downloadable through these links :

C2Hypno : <https://media.c2.care/app/last/quest/c2hypno.apk>

C2Custom : <https://media.c2.care/app/last/quest/c2custom.apk>

C2Drive : <https://media.c2.care/app/last/quest/c2drive.apk>

C2Phobia : <https://media.c2.care/app/last/quest/c2phobia.apk>

C2Addict : <https://media.c2.care/app/last/quest/c2addict.apk>

C2Nutri : <https://media.c2.care/app/last/quest/c2nutri.apk>

C2Physio : <https://media.c2.care/app/last/quest/c2physio.apk>


C2Companion : <https://media.c2.care/app/last/quest/c2companion.apk>

C2Brain : <https://media.c2.care/app/last/quest/c2brain.apk>

C2Neuro : <https://media.c2.care/app/last/quest/c2neuro.apk>

Now connect your headset to your computer via your USB cable and the port on the left side of it. Once your headset is connected to your computer, put on your headset.

You will see the following message :

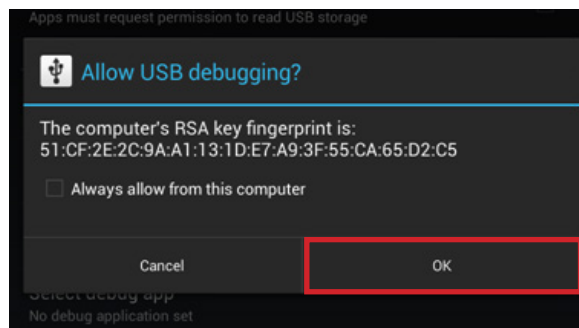


Check the **Always allow from this computer** box to stop receiving the message next time you reconnect the headset and click **OK**.

Now go back to the **SideQuest** software on your computer, you will see a green dot at the top left of the software window.




You can now click on the **Install APK file from folder** on your computer icon.

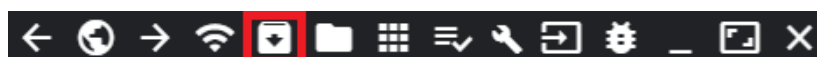

5

You will now have a window that has just been opened. You have to select the software files that you had previously downloaded. Once you have selected all the software you are interested in, press the **Open** button.

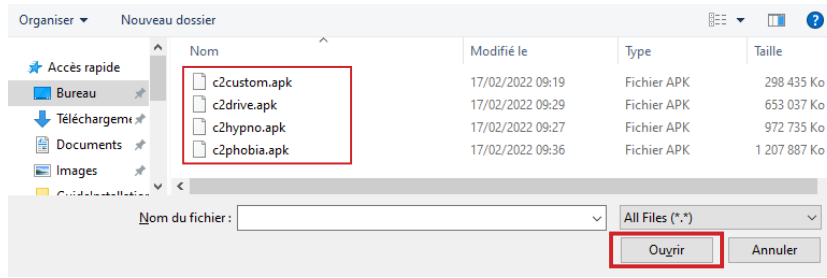

You can now unplug your headset from your computer.


IV

Launch software from the Oculus Quest

1

You will find in your library the software you have just installed.


- 1 Go to the **Applications** in the menu bar.
- 2 Go to the top right menu of the applications interface and select **Unknown sources**.
- 3 Here you can find and open all the applications you have downloaded and installed.

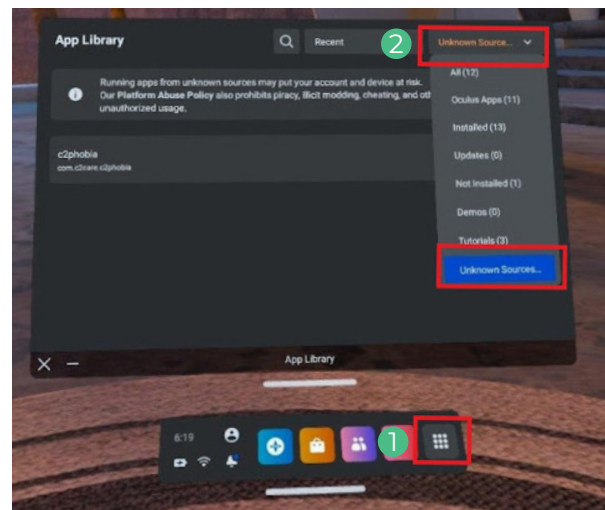

2

When you launch an application for the first time, you have to note the 5-character identification code that appears in the headset.

- 1 In your account on the website my.c2.care, tap **Add a wireless headset**.
- 2 Enter the code in the field provided.
- 3 Click on **Connect**.

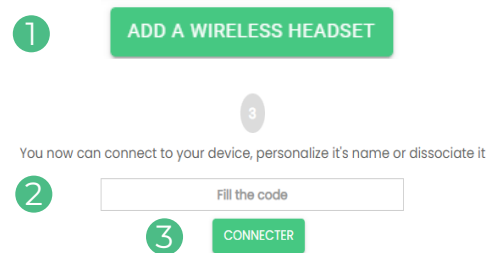

From now on you can use C2Care applications.

