



Install C2Motion Application

*This setup is compatible with: Meta Quest 2, Meta Quest Pro, Meta Quest 3, and Meta Quest 3S.
If your headset is new, complete the initial Meta setup first.*

1. Make sure your headset is up to date (Settings > General > Software Update).
2. Open the **META menu** and launch the internet **browser** from the taskbar.
3. Manually enter the following URL : <https://c2.care/motion> : The download will start automatically.
4. Open the **Library**, then **Files**, then **Recent** to locate the downloaded file.
5. Click the three dots next to the file, select "**Open with**", then choose "**Package Installer**".
6. Accept the security prompt and allow installation from unknown sources. Confirm the installation.
7. The C2Motion application is now installed. You can find it in your **Library** under "**Unknown Sources**".
8. Open the C2Motion application, note the pairing code displayed, then go to <https://app.c2.care>, log in to your account under "**Profile**", and click "**Pair a headset**" to enter the code and complete the setup.

Congratulations, your installation is complete!
You can now fully enjoy C2Motion services.

(If you encounter any technical issues, you can contact us at support@c2.care)