

C2CARE TUTORIAL

Wired headset with PC application




Do not expose the headset lenses to the sun.

- 1 Connect your wired headset to the computer using 2 cables: the one with a red dot to the port with a red dot, repeat for the blue one. Be sure to properly connect the cable to the headset. Connect the computer and the headset to the power supply.




- 2 Once the headset is plugged in, the *Mixed Reality Portal* will launch.

Turn on the controllers by pressing and holding the power button .



Mixed reality portal
Application

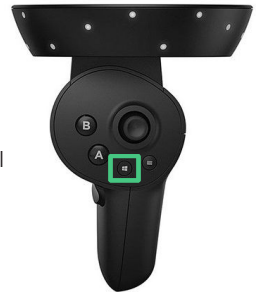

- 3 Adjust your headset on your head. Follow the instructions in the headset to set a play area.

- 4 Launch the **MyC2Care** application on your PC and log in with your credentials to access C2Care environments.



Authentication

Identifiant

Mot de passe

Connexion automatique

Connexion

Mot de passe oublié ?