




# C2CARE TUTORIAL

## Pico Neo 2 & 3 Wireless Headset



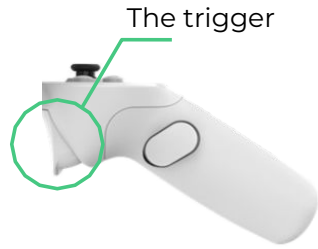

Do not expose the headset lenses to the sun.





- 1 Turn on your headset by pressing the "power" button  with a red sticker  until the logo appears. Turn on the controllers with .

If the headset or controllers do not turn on, please charge them or change the batteries in the controllers.


- 2 If you are not connected to Wi-Fi on your headset:

- 1 Click with **the trigger** on the network logo .
- 2 Click on the slider to switch it on .
- 3 Click on the chosen network and enter the password.

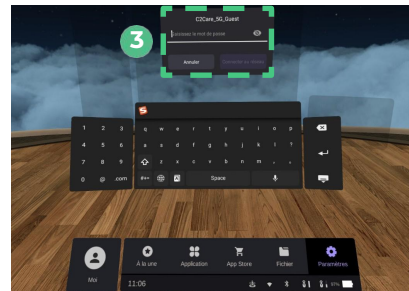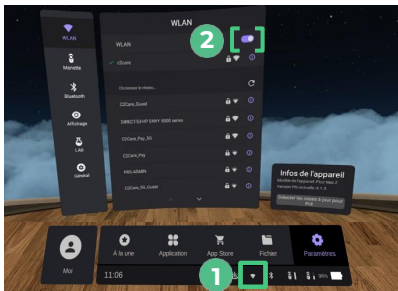

**3** Adjust your helmet on your head. Set a custom play area (first time use):

- 1 Use the trigger to click on the Custom Mode.
- 2 Proceed by following the steps and confirming with the trigger.

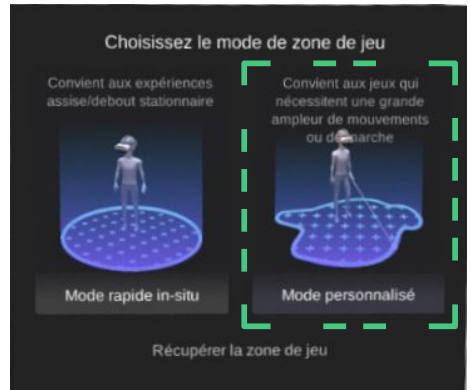

**4** Click on : [Application](#) or [Library](#) and then on a C2Care application.



**5** Launch the Google Chrome browser on your PC or tablet. Go to the [my.c2.care](#) website and log in with your credentials to control the exposures launched in the headset.

